


## PRODUCT PROFILE

NAME OF PRODUCT:	IPS Empress® IPS Empress® 2
MANUFACTURER:	Ivoclar Vivadent, Inc.
BRIEF DESCRIPTION:	Empress - pressed Leucite-Reinforced ceramic coping Empress 2 - pressed Lithium Disilicate ceramic coping
BENEFITS FOR DOCTOR'S OFFICE:	Esthetics - most translucent of all ceramic crowns
BENEFITS FOR PATIENT:	Esthetics - most translucent of all ceramic crowns
WHAT TO SEND LABORATORY:	Impression of prepared tooth, with opposing and bite registration Shade
INDICATIONS:	Empress - Anterior Crowns, Veneers, Inlays, Onlays Empress 2 - Anterior Crowns, some Posterior Crowns, Anterior 3-unit Bridges Note: Bridge pontic space must not exceed 11 mm. anteriorly or 9 mm premolar area. Bridge connector sizes must not exceed 4X4 mm anteriorly and 5X4 mm. premolar area.
CONTRA-INDICATIONS	Do not use where high strength is needed Do not use with bruxers or clenchers. Not recommended for restorations with subgingival margins.

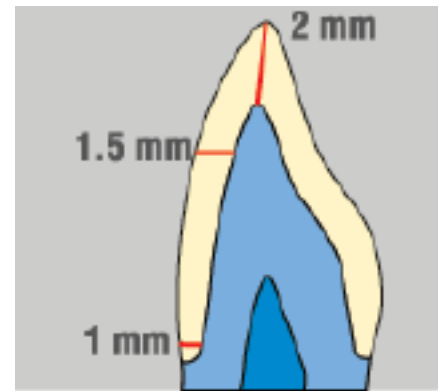

PREPARATION INSTRUCTIONS:

- 1.0 mm chamfer margin
- 1.5 - 2.0 overall reduction
- 1.5 - 2.00 occlusal reduction
- .6-.8 mm facial reduction for veneers

DAYS IN LAB:

8

SEATING INSTRUCTIONS:

- Adjust only after fully bonding (cementing)
- Dual-Cure Resin cement
- Light-Cure Resin on veneers
- Do not cement with glass ionomers or resin-reinforced glass ionomers.

AFTER CARE:

Do not bite on hard objects or chew ice.

INSURANCE CODE:

- D2740 crown - porcelain/ceramic substrate
- D2962 labial veneer (porcelain laminate) - laboratory
- D2610-2644 inlays and onlays, 1,2,3 or more surfaces
- For bridges:
- D6742 crown - porcelain/ceramic
- D6245 pontic - porcelain/ceramic

CHAIRSIDE INSTRUCTIONS:

Do not adjust extra-orally before cementing.

## TESTIMONIALS & QUOTES:

“More than ten years of clinical use and numerous scientific studies have shown that IPS Empress is excellently suitable for the fabrication of anterior restorations, which leave nothing to be desired with regard to esthetics and functionality.”

*Dr. Robert Winter, Newport Beach, CA  
Faculty, Graduate Prosthodontics, University of Washington*

## CLINICAL TESTING:

## UNIVERSITY REPORTS

Prescribed and tested at numerous North American Centers of Research Excellence:

University of Alabama  
Loma Linda University  
University of Iowa  
University of Florida  
Louisiana State University  
Tufts University  
University of Minnesota  
SUNY of Buffalo  
Ohio State University  
University of Toronto  
University of Texas  
Marquette University  
University of Washington  
University of Maryland  
University of North Carolina  
University of Michigan  
Boston University  
UCLA  
Oregon Health Sciences University  
University of Medicine and Dentistry of New Jersey