

PRODUCT PROFILE

NAME OF PRODUCT:	Lucitone FRS™ (Flexible Denture Resin)
MANUFACTURER:	Dentsply Prosthetics
BRIEF DESCRIPTION:	A flexible and monomer-free thermoplastic dental resin
BENEFITS FOR DOCTOR'S OFFICE:	More impact and stain resistant than conventional thermoplastics Easier to grind and polish Matches Lucitone shades Tissue shaded or clear clasps available Monomer and metal free
BENEFITS FOR PATIENT:	Aesthetic metal free and acrylic free denture base (for patients allergic to monomer denture resin)
WHAT TO SEND LABORATORY:	Full arch impression (custom tray) and bite registration.
INDICATIONS:	Temporary partial denture Small to medium size full removable dentures Occlusal splints and nightguards.
CONTRA-INDICATIONS	Not intended for long term use or to replace metal cast partials. Unilateral partials



- PREPARATION INSTRUCTIONS:** Standard procedures except that the flexible clasp material is able to engage a deeper undercut than a cast chrome clasp. The occlusal rest preparations cannot be utilized so a lingual blanket major connector must be used to insure adequate stability during function.
- DAYS IN LAB:** 5 days for Full or Partial Denture
3 days for Occlusal Splints or Treatment Partial
- SEATING INSTRUCTIONS:** LuciSof burs are recommended for chairside adjustment. Coarse pumice for finishing, Tripoli for polishing
Do not finish to less than 2 mm thickness.
- AFTER CARE:** Clean the denture gently with a toothbrush or denture brush using baking soda, soap, denture paste or tooth paste and water.
- INSURANCE CODE:** D5225 Maxillary partial denture - flexible base
(including any clasps, rests and teeth)
D5226 Mandibular partial denture - flexible base
(including any clasps, rests and teeth)
- CHAIRSIDE INSTRUCTIONS:** Routine Care

TESTIMONIALS & QUOTES:

“Better Impact and Stain Resistance vs. Valplast[®] and
Flexite[®] Supreme”
Journal of Prosthetic Dentistry 1999; 82:291-300

CLINICAL TESTING:

UNIVERSITY REPORTS