


PRODUCT PROFILE

NAME OF PRODUCT:	Lucitone® Denture Base Resin for Complete Dentures, Partial Dentures
MANUFACTURER:	Dentsply Prosthetics
BRIEF DESCRIPTION:	Methyl methacrylate heat and auto processed acrylic resins
BENEFITS FOR DOCTOR'S OFFICE:	Strength / Highest Impact Resistance
BENEFITS FOR PATIENT:	Beauty - are available in a number of gingival shades Strength - 2 year guarantee against breakage
WHAT TO SEND LABORATORY:	Full Arch Impression of both arches, preferably with a custom tray technique.
INDICATIONS:	Heat Processed - for denture and partial bases Auto Processed - for denture repairs
CONTRA-INDICATIONS	Patient with known allergies to methyl methacrylates (acrylic resins)

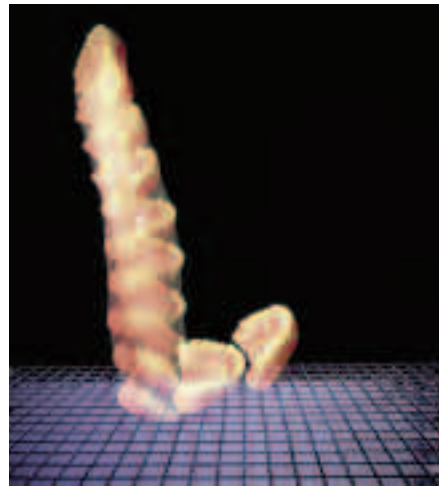

PREPARATION INSTRUCTIONS:

Full arch impressions (custom tray) for master models
Occlusal Rims / Baseplates - modified for vertical and to record bite registration
Tooth Try In - to verify vertical, bite, shade, etc.
return for processing of base

DAYS IN LAB:

After set-up, processing is 2 days

SEATING INSTRUCTIONS:

Check and adjust peripheries if needed
Minimal occlusal adjustment until denture settles in.

AFTER CARE

and

Clean the denture with a toothbrush or denture brush using baking soda, soap, denture paste or toothpaste water. You may soak dentures in a commercial denture cleanser or a solution of water and white vinegar. Periodically disinfect the denture for 10 minutes in a cup of water with 1/2 tsp bleach.
Do not soak in mouthwash - alcohol dries out the resin.

INSURANCE CODE:

D5110 - complete denture - maxillary
D5120 - complete denture - mandibular
D5130 - immediate denture - maxillary
D5140 - immediate denture - mandibular
D5211 - maxillary partial denture - resin base
(includes clasps, rests and teeth)
D5212 - mandibular partial denture - resin base
(includes clasps, rests and teeth)

CHAIRSIDE INSTRUCTIONS:

Routine