



Fixed-Detachable Denture flow chart:

First Appointment:

- * Take alginate impression.
- * Rx Requesting custom tray. (Two days in Lab.)

Second Appointment:

- * Remove healing abutments. Tools can be loaned if needed.
- * Place Impression copings. Tighten screw finger tight.
- * Take open tray impression for master cast.
- * Replace healing abutments.
- * Rx requesting Base plate and Wax Rims. (Four days in Lab.)

Third Appointment:

- * Use verification jig to check fit.
- * Take Bite registration.
- * Rx requesting set-up. (Three days in Lab.)

Fourth Appointment:

- * Verify Set-up.
- * Rx requesting Bar fabrication (Fifteen days in Lab.)

Fifth Appointment:

- * Try-in Bar for passive fit with set-up
- * Return for process. (Two days in Lab.)

Sixth Appointment:

- * Seat Bar and torque to proper specifications. Tools can be loaned if needed.
- * Plug access holes. (Composite)